

VORTEX®

Short instruction




Instructions for Use (for use at home)



Carefully read this entire manual before the VORTEX® is used

1

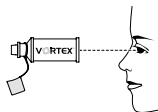

- Before every use, inspect the VORTEX® for damage and make sure there are no foreign bodies in.

2




- Remove the sealing cap of the spray / MDI (= Metered Dose Inhaler).
- Shake the spray vigorously. Observe the operating instructions for your spray.

3




- Insert the mouthpiece of the spray into the blue connecting ring of the VORTEX®.
- Pull the protective cap off.

4

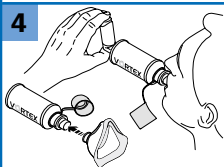

- Place the mouthpiece between your teeth and enclose it with your lips. Alternatively, place the VORTEX® with mask gently on the face. Mouth and nose must be covered.

5

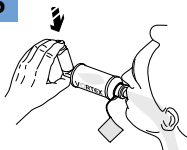


Using VORTEX® with mouthpiece

- Trigger the medication pull and take a deep, slow breath.
- Hold your breath briefly.
- Breathe out slowly into the mouthpiece.

Ensure that only one pull is sprayed into the VORTEX® at a time.

6

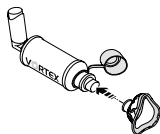

Using VORTEX® with mask

- Make sure the mask fits tightly against the face.
- Slowly breath in the medication.
- For children under 18 months, hold the mask tightly against the child's face for 4-6 breaths and for children over 18 months for 2-4 breaths.

7



If you need more than one medication pulls

Wait shortly, then repeat steps 4 and 5 (or 6) in the instructions.

VORTEX®

Short instruction




Cleaning

Be sure to clean VORTEX®

- 🔄 The VORTEX® must be cleaned thoroughly before the first use.
 - 🔄 If contamination is visible VORTEX® must be cleaned immediately.
- The VORTEX® can be cleaned as follows (see Figure 8-13)

8

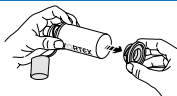

- 🔄 Remove the connecting ring and mask (if present). Do not remove the mouthpiece and the protective cap.

9




- 🔄 Thoroughly clean all parts thoroughly in warm tap water and dishwashing liquid.

10



- 🔄 Rinse all parts thoroughly under clean running water and then shake to remove excess water.

11

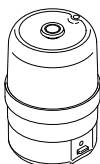

Disinfection


- 🔄 **Recommended method**
With vaporisator or disinfectant.

12



- 🔄 **Alternatively, in boiling water**
Place the individual parts of the VORTEX® and the accessories (e.g. mask) in boiling water for at least 5 minutes. Use a clean pot and fresh drinking water and ensure that there is sufficient water in the pot.

13

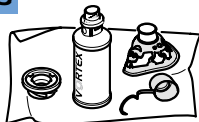

Drying

- 🔄 The VORTEX® and the accessories must be dried completely. Due to its antistatic properties it can also be dried with a clean tea towel.

Distributör: Active Care Sverup AB

Uggledalsvägen 47 · 427 40 Billdal
Tel: 031-917525 · Fax: 031-917505
E-post: info@activecare.se
hemsida: www.activecare.se



PARI GmbH
Spezialisten für effektive Inhalation
Moosstraße 3 · D-82319 Starnberg

