

# USER INSTRUCTIONS FOR RESPIMAT® INHALER

ENGLISH

## DAILY USE

You only need to use this inhaler **once a day**. Each time you use it, take **two puffs**.



### 1. TURN

- Keep the cap closed
- **TURN** the clear base in the direction indicated by the arrows on the label until you hear a click (half a turn)



### 2. OPEN

- Completely **OPEN** the cap



### 3. PRESS

- Breathe out slowly and fully, and then close your lips around the mouthpiece
- Breathe in slowly and deeply through your mouth and **PRESS** the dose-release button. Keep breathing in slowly
- Repeat **TURN, OPEN, PRESS** for a total of two puffs

## WHEN YOU NEED TO REPLACE THE CARTRIDGE OR INHALER

The dose indicator shows how many puffs remain in the cartridge.



60 puffs remaining.



Fewer than 10 puffs remaining. Obtain a new cartridge.



The cartridge is empty. Turn to remove the clear base. Pull the cartridge out of the inhaler. Insert a new cartridge.

Once you have used six cartridges in the inhaler, you must obtain a new Respimat inhaler pack.

## INITIAL SETUP



### Remove the clear base

- Keep the cap closed
- Press the safety catch while pulling off the clear base with your other hand



### Insert the cartridge

- Insert the cartridge into the inhaler
- Place the inhaler on a firm surface and push down until it clicks into place



### Make a note of the cartridge

- Tick the checkbox on the inhaler label to keep track of the number of cartridges
- Push the clear base back into place until you hear a click



### TURN

- Keep the cap closed
- Turn the clear base in the direction indicated by the arrows on the label until you hear a click (half a turn)



### OPEN

- Completely open the cap



### PRESS

- Point the inhaler down at the ground
- Press the dose-release button
- Close the cap
- Repeat steps 4-6 until a mist is visible
- Then repeat steps 4-6 three more times
- Your inhaler is now ready to use and contains 60 puffs (30 doses)